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EN

BASIC PRESCRIBING INFORMATION FOR

Rubber bags for ice

Purpose. Rubber bags for ice multi-purpose are intended for aerial cooling down of the body and are reusable. Bags for ice meet requirements of DSTU 2909-94 (GOST 3302-95) and are to be considered operable. Depending on sizes the bags for ice could have three different numbers such as 1, 2 and 3.

Scope of implementation. Sanitary and hygiene as well as for medical institutions and domiciliary use.

Bacterial purification. Bacterial purification of the bags for ice in medical institutions is carried out in 3 % by weight solution of oxygenated water or in 1 % by weight solution of chloramine by the method and procedures in accordance with actual normative documents. Domiciliary bacterial purification of the product after use is in thorough washing of bulb with hot flowing water.

Implementation. Determination of indication for use, choosing of required size of bags for ice, and recommendations regarding subsequent use are carried out by attending medical doctor. The bags for ice are to be washed with warm soapy water before use and rinsed with flowing water.

Storage. Bags for ice are stored at the temperature of from 0°C to 25°C and relative humidity of air not more than 80 % at the distance of not less than 1 m from heat irradiating appliances. During storage the bags for ice should not be exposed to influence of oils, petrol and other substances destroying rubber.

Manufacturer's warranties. Manufacturer warrants the quality of bags for ice in case of adherence of storage and operation conditions. **Storage warranty period** is 3.5 years from day of production.

Guaranteed service life is 3 years from the day of purchase through the retail trade network and is 1.5 years from the day of receipt by consumer for medical institutions.

Authorised Representative's in EU, Name and Address:

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MEDEX LLC, Wadowicka Str., 6/06, 30-415, Cracow,