RUS

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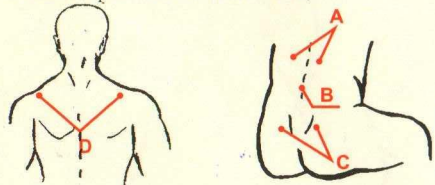
ENG

**RADICULITIS**

**Point A** – is 3 cm to the side of the spine, on the level of the space between the first and the second lumbar vertebrae.

**Point B** – is a single point between the second and third lumbar vertebrae, i.e. on the waist. Pressing on the point is also helpful at abnormal menstrual periods.

**Point C** – is 3 cm to the side of the spine, on the level of II-III sacral vertebrae. It is mainly the middle of the sacral bone, which consists of five adjoined vertebrae, the second of them comes slightly about, if you sit and put your arms along the body.

**PARASYNOVITIS OF THE SHOULDER JOINT (FROZEN SHOULDERS)**

**Point D** – is on the upper part of the shoulder, above the mid-clavicalia. You should press the point with the massager "Wonder Finger" in the direction of the rest of the fingers.

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ENG

**INSOMNIA**

**Point A** – is a single point on the crown of head, close to the head top in the hollow on the «parting in the middle» - where there is a vertical line, starting from the ear through the top points of the rims of the ear.

**Point B** – is behind the ear, 1-2 cm above the border of hair over the spine, below the muscle, close to the bone.

**Point C** – is on the back surface of arm, above the wrist, approximately 3 cm in the middle between chordae.

**Point D** – is on the palm surface of the hand, is the inside of the previous point, located between 2 tendon bands, 1.5 cm from the radiocarpal fold.

**MOTION SICKNESS**

**Point E** – is in the centre of the receding part of the shoulder. *Acupressure (point massage) comes from the ancient Chinese doctors. It can not only cure the diseases but also stimulates body defences, which are necessary to prevent diseases. Masager "Wonder-Finger" was created for simplification of acupressure technique.*

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ENG

In contrast to acupuncture point acupressure points can be easily found on the body – these are small areas on the human skin, they are not intended for needle therapy; so if they are shifted slightly, it does not really matter much.

The advantages of acupressure are evident:

- No age limitations;
- No side effects;
- No need to buy expensive devices;
- No fear of the patient for the treatment;
- No drug dependence;
- High efficiency of the present technique;
- Acupressure may be combined with other treatment methods.

When you massage problem areas of the body by pressing them with circular movements using Masager "Wonder-Finger", you may get general massage, relieve tension and muscle pain.

The present specification includes only a small set of acupoints, which are more often use in action. The degree of pressure and intensity of movements is defined individually.

Manufacturer: "Torg Lines" LLC, Russia 125299, Moscow, Kosmonavta Volkova str., 7/1 Produced in accordance with GOST Р 50444-92 TU 9398-001-52751052-2011 Registration Certificate: №FSR 2008/02297 dated V4 24, 2011 The product is patented. Shelf life 3 years. Warranty period: 3 year from the day of manufacturing.

**Contraindications:**

Massage is not recommended at the presence of foul and open wounds; acute inflammatory processes; infectious diseases, accompanied by the increase of body temperature; and bleeding.

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